

EASY, ACCURATE RESULTS

LEUKO EZ VUE®

Only rapid test to detect intestinal inflammation from enteric infection

- Detects elevated levels of fecal lactoferrin, a marker of fecal leukocytes and an indicator of intestinal inflammation
- Simpler and more accurate than microscopy for fecal leukocytes
- Rapid results in 10 minutes



Product Order Information				
Product Name	Cat. No.	Sample Type	Time to Result	Format
LEUKO EZ VUE®	T30355	Fresh, frozen, refrigerated or room temperature	10 minutes	Lateral flow qualitative

The LEUKO EZ VUE® test is simpler and more accurate

LEUKO EZ VUE®

- Standardized and reproducible results in 10 minutes
- Clear results are unaffected by cell breakdown
- Samples can be stored up to 2 weeks refrigerated or at room temperature without loss of sensitivity
- Can be batched or run individually for on-demand testing

Microscopy

- Subjective interpretation can vary by technician
- Must be run immediately to minimize cellular breakdown
- Tedious to perform and time consuming
- Significant technical training needed

Assay Protocol




Add 2.5mL Diluent to a tube




Fill flared tip with sample and add to tube containing *Diluent*.



Add filter tip to tube. Vortex to mix.

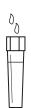





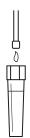



Add 5 drops to Sample Well. Read at 10 minutes.

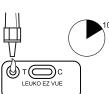

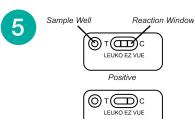

Negative

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.



© 2021 TECHLAB, Inc. All rights reserved.

LEUKO EZ VUE, the TECHLAB Logo, and TECHLAB are trademarks of TECHLAB, Inc. All other trademarks referenced are trademarks of their respective owners. PN 842021174

Distributed by



Developed and manufactured by

